

Product Bulletin

14 February 2023


ZOLL AED Plus® Duracell Battery

ZOLL continues to recommend Duracell Coppertop CR123-style batteries (DL123ABKM) batteries for use with the AED Plus[®]. However, it is important to note that Duracell now manufactures Coppertop batteries in the US and China. While the batteries may look similar, they do perform differently. To ensure optimum power longevity with the AED Plus, customers should use only batteries manufactured in the US. Batteries produced in the US include 'Made in USA' on the battery cell label. Batteries produced in China are labeled "Made in China".

Frequently Asked Questions (FAQ)

- Why can't AED Plus use any CR123 batteries?
 - o It is possible for batteries made in China to have a different chemical makeup from the ones made in the US. Duracell CR123 batteries manufactured in the US have been validated by ZOLL and one of the approved batteries listed in our device Admin Guide.
- How can I tell if my battery is the right battery?
 - Please see the image with highlighted "Made in USA"

ZOLL only distributes Duracell batteries "Made in USA". Customers are encouraged to purchase replacement Duracell batteries from the distributors you bought your AED from or directly from ZOLL. Or, verify that product is made in the USA when sourcing the product locally.

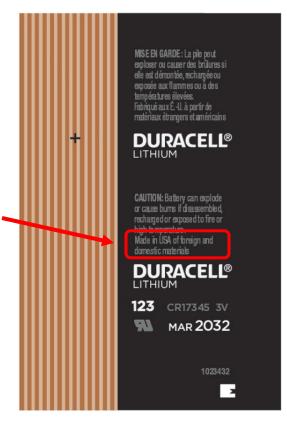

Customers	See <u>www.zoll.com/contact/worldwide-locations</u> for phone number and contact
	information.