

# SLISHMAN PRESSURE WRAP - More than bleeding control

## Instructions For Use

1. Loop over injured limb & apply to injury (or simply wrap for entrapped limb).
2. Cinch the red sleeve until snug.
3. Use a wide wrap for long lacerations, or a narrow, tight wrap for more severe bleeding (between the injury & heart).
4. Wrap the elastic all the way to the white label, then secure hooks to the elastic.
5. Note application time on the white label.
6. Adjust for pain minimization, while monitoring bleeding, pulses, sensation and motor function.
7. Leave in place until definitive wound management can be accomplished ASAP.



Scan for video instructions



[www.SlishmanPressureWrap.com](http://www.SlishmanPressureWrap.com)  
Made in Pakistan | Lot #TS202201  
US Patent No. 11,246,602

\*Do not apply to the neck.

\*\*Also useful to protect wounds, stabilize extremities, or to hold splints in place.